

PROPER FIT &
HIGH PERFORMANCE
DISPOSABLE
AFFORDABLE
COMFORTABLE



INTRODUCING BLOX


the *UnFake* Mask

REAL MASKS. FOR REAL PEOPLE.

Fit Instructions


Step 1

Separate the edges of the mask to fully open it. Slightly bend the nosepiece to form a curve.

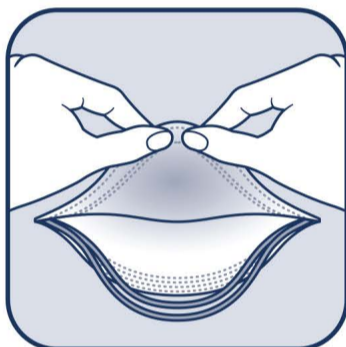

Step 2

Using your index fingers and thumbs, remove both headbands from the mask.

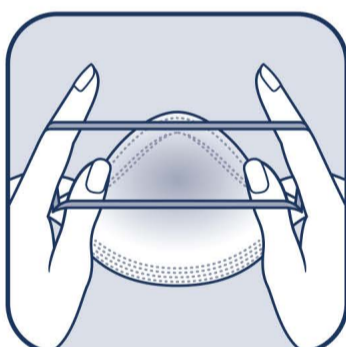

Step 3

Continue holding the headbands and cup the mask under your chin with the nosepiece facing upwards.




Step 4

Pull the headbands over your head and let them rest at the base of your neck, below your ears.



Step 5

Position the top headband to rest at the crown of your head, above your ears.

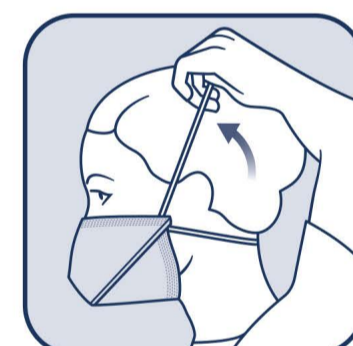

Step 6

Using both hands, firmly press down to form the nosepiece to the bridge of your nose and below the eyes. Adjust to secure all edges and perform a seal check.



✓ Performing a seal check:

It is necessary to perform a seal check prior to each time of use. To check the mask seal, cup the mask with both hands, breathe in forcefully. You should feel pressure inside the mask. If you feel any air leakage on an inhale or exhale, adjust the nosepiece and headbands until you feel a secure fit with no air leakage. Repeat the seal check again to confirm proper fit.