

BLOX-9801W, BLOX-9801B

blox⁹⁸ Particulate Filtration Face Mask

Fit Instructions


Step 1

Separate the edges of the mask to fully open it. Slightly bend the nosepiece to form a curve.




Step 2

Using your index fingers and thumbs, separate both headbands.

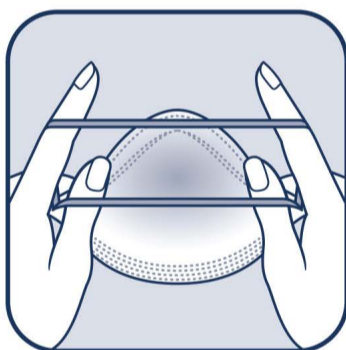

Step 3

Continue holding the headbands and cup the mask under your chin with the nosepiece facing upwards.

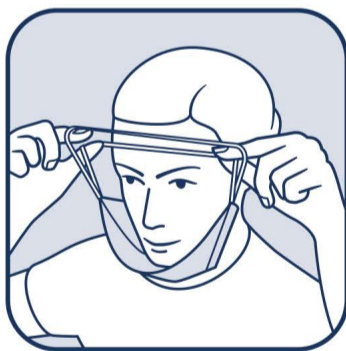


Step 4

Pull the headbands over your head and let them rest at the base of your neck, below your ears.




Step 5

Position the top headband to rest at the crown of your head, above your ears.

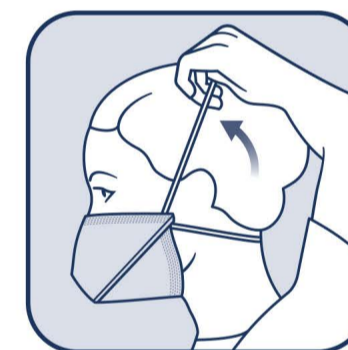

Step 6

Using both hands, firmly press down to form the nosepiece to the bridge of your nose and below the eyes. Adjust to secure all edges and perform a seal check.

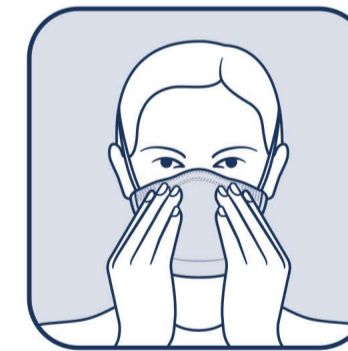

Performing a seal check:

It is necessary to perform a seal check prior to each time of use. To check the mask seal, cup the mask with both hands, breathe in forcefully. You should feel pressure inside the mask. If you feel any air leakage on an inhale or exhale, adjust the nosepiece and headbands until you feel a secure fit with no air leakage. Repeat the seal check again to confirm proper fit. DO NOT enter a contaminated area if a proper seal cannot be obtained.



Manufactured by ivWatch, LLC at our FDA registered facility
700 Tech Center Parkway, Suite 300, Newport News, VA 23606

To order or learn more, visit: blox.ivWatch.com
If you have any questions, contact: support@ivWatch.com